



## P.U. Perching Stool Range



## Fixing and Maintenance Instructions

Thank you for deciding to purchase the PU Perching Stool from SimplyMed. The PU Perching Stool range is manufactured from the finest material and components available, when installed and used correctly it is designed to give many years of reliable trouble-free service. The PU Perching Stool is available with optional arms, back or padded back.

PU Perching Stools are delivered complete and do not require assembly.

**N.B. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the chair for the particular user.**

**SMH003 Plain Perching Stool**

**SMH003A Plain Perching Stool adj height**

**SMH004 Perching Stool with Arms**

**SMH005 Perching Stool with Arms & Plain Back**

**SMH006 Perching Stool with Arms & Padded Back**

**190 kg (30 stone) weight limit.**

**DO NOT exceed the weight limit stated – doing so could put the user at risk.**

## **BEFORE INSTALLATION**

Please examine the item for any visible damage before use. If you see any damage or suspect a fault, please do not use the product, but contact your supplier immediately for support.

Please dispose of all packaging carefully.

## **HEIGHT ADJUSTMENT**

Remove the 'E' clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see fig. 1)

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and that it fits neatly and snugly around the outside leg. (see fig. 2).



*Note the perching stool is designed to slant to the front. Do not adjust any of the legs to make the seat level as this will cause excessive stress to the legs compromising user safety and possibly leading to permanent damage of the perching stool, including fracture of the tubular and other sections.*

## **COMPOSITION**

The perching stools are made from mild steel PPA/epoxy coated with upholstered CMHR foam and fire retardant vinyl seats or contoured polyurethane seats.

## USE OF THE PERCHING STOOL

When moving always lift the perching stool by placing your hands on the tubular metal frame beneath the seat.



Do not allow children or unauthorised persons to play with the equipment or use without proper supervision.



Do not rock back and forth or sideways when using the stool as this will cause excessive stress and possible damage and failure which may cause injury.



Do not use the perching outside the home or on uneven surfaces (such as in the garden) as this will cause stress to the legs and possible damage which may cause injury.



Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury.

## CLEANING

Prior to cleaning remove all pin clips and the four extension legs with the ferrules fitted. Clean these items separately to the main frame.

Clean the upholstery using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads will seriously damage the upholstery beyond repair.

The products in this group can be treated with formaldehyde (upholstery only), but it is important that this treatment be carried out by an approved contractor or an experienced person and under suitable conditions.

## REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.


This includes:

- PAT test for electrical items
- Tightness of all nuts/bolts/castors
- Other screw in/bolt in/push in components.
- Also check all upholstery for security, splits, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

When reissuing, a risk assessment may be required as to the suitability of the chair for a particular user.



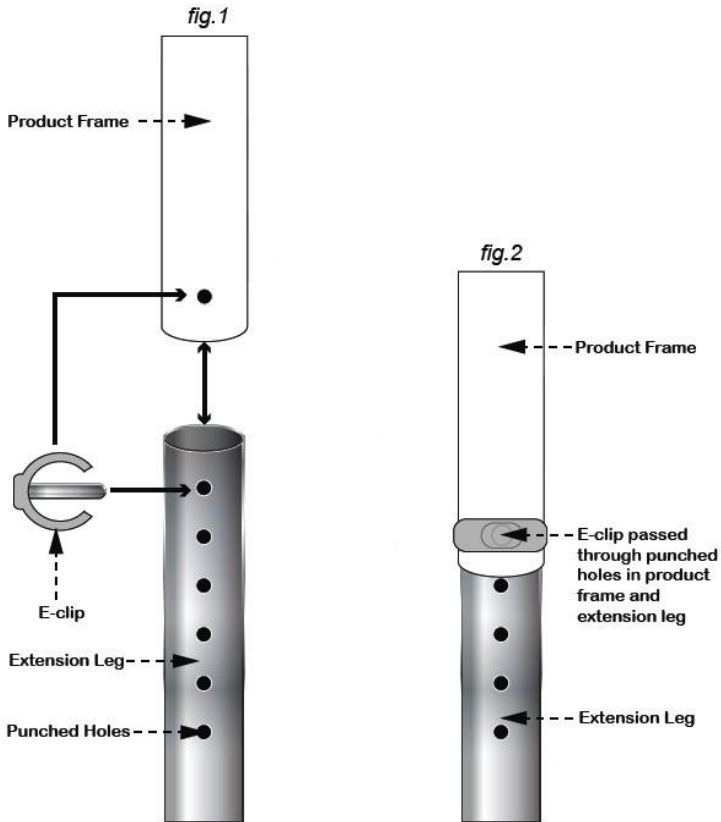

## IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by SimplyMed Limited, or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.





**SimplyMed Ltd**  
**Unit B, New Venture Business Park**  
**Leopold Street, Pemberton**  
**Wigan, WN5 8FR**  
**Tel: 01942 243 999 | Fax: 01942 217992**  
**email: [info@simplymed.co.uk](mailto:info@simplymed.co.uk)**  
**[www.simplymed.co.uk](http://www.simplymed.co.uk)**